


QUICK REFERENCE INSTRUCTIONS

2019-nCoV Ag Rapid Test Kit (Immunochromatography)

SAMPLE REQUIREMENTS


- 1 Insert a sterile swab into the nostril of the patient, reaching the surface of the posterior nasopharynx.
- 2 Swab over the surface of the posterior nasopharynx. Hold the swab in the nose for 15-30s and rotate it 3 times.
- 3 Withdraw the sterile swab from the nasal cavity.

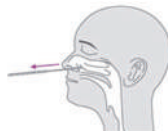

***Note:**


1. The samples should be used as soon as possible after collected (within half an hour).
2. Samples should not be inactivated.

SAMPLE PROCESSING

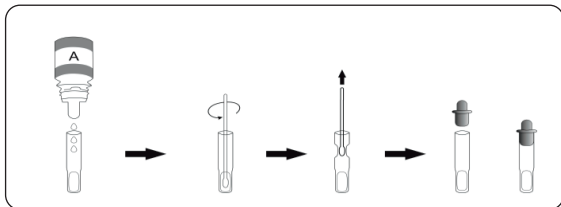

- 1 300 μ L (to the 300L mark line,9 drops)Sample Buffer.
- 2 Vigorously mix at least 10 times.
- 3 Squeeze liquid from swab.
- 4 Cover the dropper head.


TEST PROCEDURE

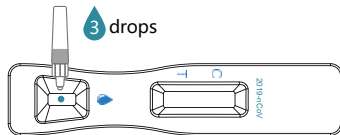

3 drops to the each well

NOTE: Take the required reagents and test cards to equilibrate to room temperature.

INTERPRETATION OF TEST RESULTS

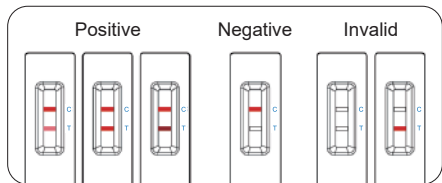

Read the result after 15mins. less than 15 minutes may lead a false negative result; more than 15 minutes may lead a false positive result.

IMPORTANT

1. Read instructions carefully before beginning.
2. All specimens must be at room temperature before testing.
3. Check expiration date on each individual test package or outer box before using.
4. Do not use any test past the expiration date on the label.
5. Disposable gloves and safety glasses are recommended when running this test.
6. Wash hands thoroughly after handling any patient sample.



Huachenyang (Shenzhen) Technology Co., Ltd
Area A, F8/Bldg 4, Hengchangrong High Tech Park, Shangnan East
Road, Hongtian, Shajing, Bao' an, Shenzhen, 518125, Guangdong, CHINA

TEL: +86(0)755-27393226
info@chenyanglobal.com
www.chenyanglobal.com